



**DIRECTIONS FOR USE:
INTELLIEAR®/INTELLIEAR® PLUS**

IMPORTANT:

Before using the IntelliEAR® PLUS for the first time, you must charge the internal rechargeable battery for at least twelve hours. Please refer to Page 15. of this guide, regarding the Rechargeable Battery, for assistance.

© 2015 KAGAN UNLIMITED

IntelliEAR® instructions and its related logo are registered trademarks of KAGAN UNLIMITED

Chicago, Illinois, 60625

All Rights Reserved.

Welcome to the Sound Revolution!

You are now the owner of a IntelliEAR®, the revolutionary sound amplification system that is lightweight, compact, and discreet.

When used correctly, the IntelliEAR® promotes a happier, healthier lifestyle by allowing you to hear while maintaining your normal routine and activities.

Please read the following instructions through carefully before using. Thank you for purchasing the IntelliEAR®.

www.intelliear.com

IntelliEAR®

- 5. IntelliEAR® (button cell model)
- 6. Product Specifications
- 6. Contents of Package
- 7. Directions for Use
- 7, 8. Battery Installation
- 9,10. Ear Tip Installation
- 11. Switch ON, Volume Controls
- 11. How to Wear/Adjust Ear Amplifier

FOR BOTH MODELS:

- 18-20. Frequent Questions
- 21. Ordering Replacement Parts
- 22. Refund and Return Policy

IntelliEAR® PLUS

- 12. IntelliEAR® PLUS (lithium battery)
- 13. Product Specifications
- 13. Contents of Package
- 14. Directions for Use
- 14. On/Off
- 15. Rechargeable Battery
- 16. Product Diagram
- 17. Volume Control
- 17. How to Wear/Adjust Ear Amplifier

Additional information at:
www.intelliear.com

InteliEAR[®]



With replaceable button-cell batteries.

INTELLIEAR® SPECIFICATIONS

- Input voltage: 3.0V (AG13 button cell battery 1.5V x 2)
 - Operating Current: 10mA
 - Speaker Impedance: 200Ohm
 - Gain: 50dB+/-6dB
 - Output Power: 3mW
 - Amplifier frequency response: 200-5000Hz
 - Operation Time: 12 hours continuously
(when using AG13 batteries)
-

CONTENTS OF PACKAGE

- (1) IntelliEAR®, button-cell model
- (3) Soft Ear Piece Covers
- (2) AG13 batteries in unit
- (6) AG13 replacement batteries

Directions for Use

INSTALL BATTERY

Step 1:


Open the battery cover and install the two AG13 batteries. Please note the polarities indicated:



INSTALL BATTERY (CONTD.)

Step 2:


When batteries are in place, close cover.

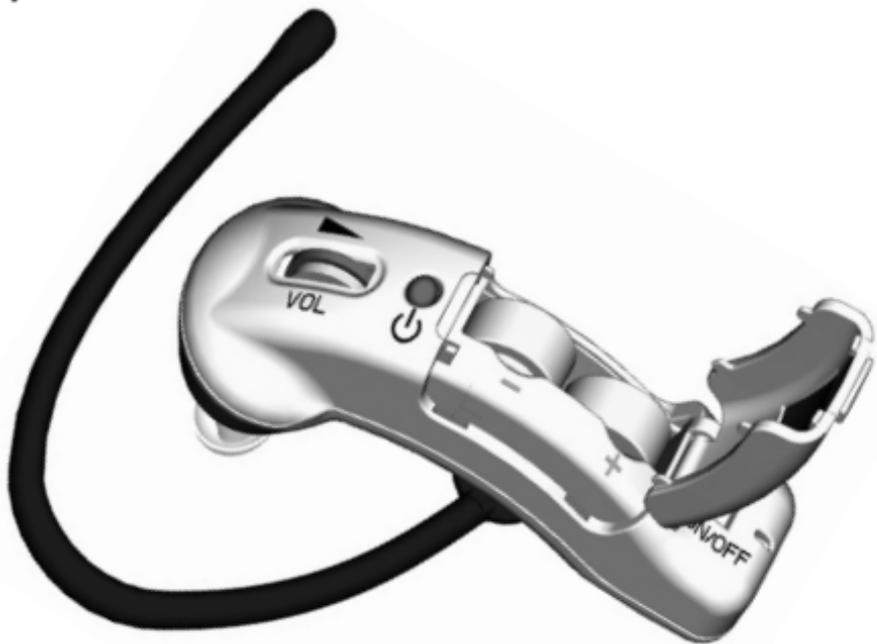

INSTALL EAR TIPS

The IntelliEAR® comes with 3 silicone ear tips that have different head diameters for different size ear canals. We suggest you try all 3 sizes for the most comfortable fit and best listening results.


RIGHT WAY:

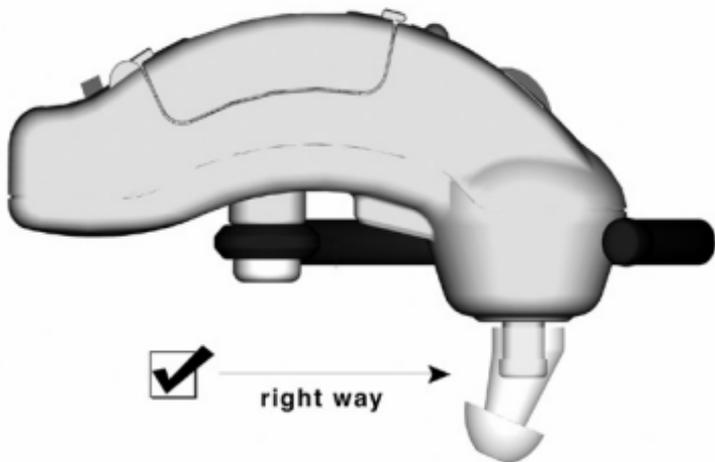

INSTALL EAR TIPS (CONTD.)

Place the silicone ear tip over knob, with the tip pointing towards the front of the unit. See diagram at left marked **RIGHT WAY**. Do not place the ear tip the **WRONG WAY**.

WRONG WAY:

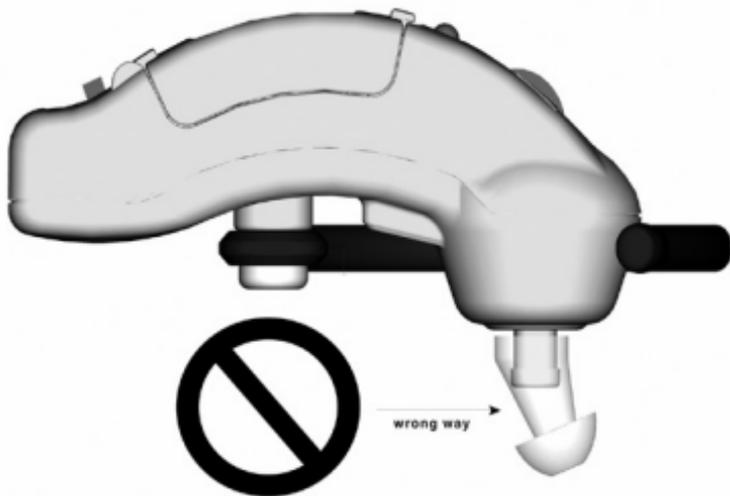

SWITCH “ON” THE UNIT AND SET VOLUME

The switch is located on one side of the unit. When the switch to ON position, the red light comes on. Slide it to the OFF position and the red light turns off. Turn the knob to change the volume of the unit. Once the volume control is set at the desired level for your personal listening preference, you only need to slide the power switch to the ON position and you can enjoy the same sound level each time.

HOW TO WEAR THE INTELLIEAR®

Wrap the rubber wire over your ear until the unit is secure and comfortable. This rubber wire is made to easily adjust to all ear shapes and sizes. The IntelliEAR® is packed to fit on the right ear. To use it on the left ear, bend the wire to the opposite side; adjust ear tip angle for left ear.



IntelilEAR[®] PLUS

With rechargeable Ni-MH battery.

INTELLIEAR® PLUS SPECIFICATIONS:

- Power Supply: Rechargeable Battery
 - Input Voltage: 2.4VDC
 - Operating Current: $\leq 10\text{mA} \pm 20\%$
 - Speaker Impedance: 200 Ohm
 - Gain: 50dB \pm 6dB
 - Output Power: 3mW
 - Unit Weight: 20g (*Including rechargeable battery.*)
-

CONTENTS OF PACKAGE

- (1) IntelliEAR® PLUS
- (3) Soft Ear Piece Covers
- (1) AC Charger, Rechargeable Nickel-Metal Hydride Batteries

Directions for Use

PRIOR TO FIRST USE

Remember to charge the IntelliEAR® PLUS for a minimum of 12 hours prior to first use! See section titled “Recharging the Battery” on the following page for help.

SWITCH “ON” THE UNIT


The ON-OFF Power switch is located on one side of the unit. When you slide the switch to ON, the red light comes on. Slide it to the OFF position and the red light turns off. Be sure to turn the unit OFF when you recharge. The battery cannot be recharged when the unit is ON.

RECHARGEABLE BATTERY

A rechargeable Ni-MH battery operates the IntelliEAR® PLUS unit. *Before* using the unit for the first time, you must charge the internal Ni-MH rechargeable battery for a minimum of **twelve hours**. If the sound is weak while the volume control is at the maximum position, the battery should be recharged immediately.

RECHARGING THE BATTERY

Do not remove the Ni-MH battery from the IntelliEAR® PLUS unit. The AC adapter included with this unit is for recharging the battery. Locate the DC jack on the inside of the unit (*diagram at right*). To recharge the battery, insert the DC plug into the DC jack and plug the AC adapter into a wall socket. A green light will come on to indicate that the battery is being charged. A full recharge process takes around 12-14 hours. Only use the AC adapter supplied by the manufacturer (see page 17).

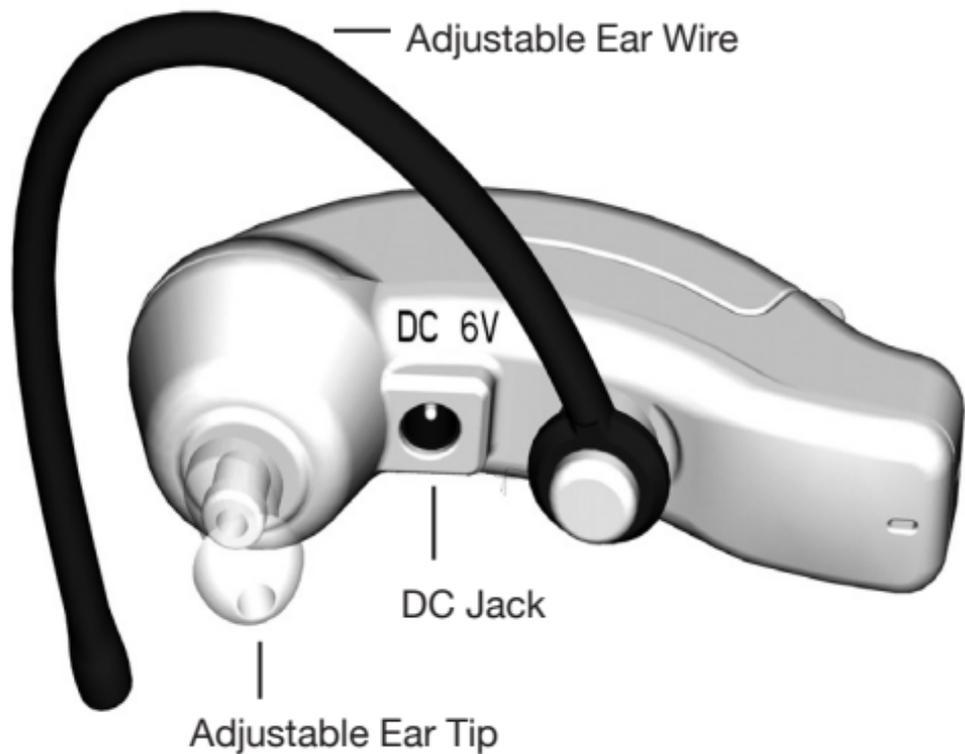

***CAUTION:** Only use the AC adapter that has been supplied by the manufacturer. The manufacturer does not refund or service items damaged by unauthorized adapters.

VOLUME CONTROL

Volume control is located on the side of the unit. Turn the knob counter-clockwise to increase the volume. Turn the knob clockwise to decrease the volume. Once the volume control is set at the desired level for your personal listening, you need only slide the power switch to the ON position and you can enjoy the same sound level each time.

HOW TO WEAR THE EAR AMPLIFIER

Wrap the rubber wire over your ear until the unit is secure and comfortable. This rubber wire is made to easily adjust to all ear shapes and sizes. If desired, the soft ear tip cover can be taken off and cleaned using lukewarm water. Only clean the soft ear tips in this way. Do not submerge the IntelliEAR®/PLUS in water.

FREQUENT QUESTIONS (BOTH MODELS)

1) Why do I need to charge the battery before the first use?

The IntelliEAR® PLUS uses a Ni-MH battery. These batteries require a full charge before the first use of the product for the batteries to function at their optimal use. The IntelliEAR™ PLUS can be charged for 12 hours or overnight. Button-cell batteries for the IntelliEAR™ last approximately 12 continuous hours.

2) How do I know if the battery is fully charged?

Charging the IntelliEar® PLUS 12-14 hours can fully recharge the internal battery (*see page 15*).

3) Will the green light turn off when battery is fully charged?

No, the light will stay on when fully charged and the unit is in the off position. When the unit is turned on again, the red light will appear.

4) How can I get it to fit in my ear properly?

Both the IntelliEAR® and IntelliEAR® PLUS come with three different style ear tips. Choose an ear tip that fits comfortably in your ear but is not too loose. Use the flexible foam earpiece to wrap around the outer ear to hold the unit at the desired position.

5) How do I clean the IntelliEAR®/PLUS?

Wipe the unit with a clean, damp cloth. Do not immerse the unit in water or it will destroy the battery and possibly cause electric shock.

6) How do I clean the IntelliEAR®/PLUS soft ear tips?

The ear plug cover can be removed and cleaned with lukewarm water. Do not use soap or detergents. Clean ear tips in a bowl of warm water and wipe clean with a cloth.

7) Why is there a high pitched noise?

Audio feedback may be caused when the IntelliEAR® and

IntelliEAR® PLUS earpiece is exposed to its own highly sensitive microphone. Feedback is normal, however, repeated and/or prolonged occurrences should be avoided. To prevent audio feedback, remember to switch OFF the IntelliEAR®/PLUS before placing it into, or removing it from, your ear.

8) Why is the unit not holding a charge?

The IntelliEAR® PLUS will hold a charge up to 5 hours. If your unit is not holding the charge, it could be because it was not properly charged the minimum 12 hours prior to its first use.

9) Will the IntelliEAR™/PLUS work with my cell phone?

The IntelliEAR®/PLUS is an audio amplification device. While cleverly disguised as a cell phone earpiece, the unit is not intended to be compatible with any phones on the market and does not work in conjunction with cell phones.

Visit www.intelliear.com for more information.

Item	Cost	Shipping	Total
3 pack Ear Tips	\$2.00	Free	\$2.00
42 pack cell batteries	\$9.95	\$6.95	\$16.90
IntelliEar™	\$19.95	\$6.95	\$26.90
IntelliEar™ PLUS	\$29.95	\$6.95	\$36.90
AC Adapter	\$19.95	\$6.95	\$26.90

*Include a check or money order and a letter specifying the product replacement you require to **KAGAN UNLIMITED:**

IntelliEAR® Replacement Parts

c/o KAGAN UNLIMITED, 3336 W LAWRENCE AVE.

SUITE 301, Chicago, IL 60625

REFUNDS & RETURNS

To receive a credit or a refund for faulty IntelliEAR®/PLUS unit, you must return that unit, with proof of purchase, where the product was purchased. Only the company which sold you the product is able to refund or credit your account. Authorized sellers of the IntelliEAR®/PLUS have the right to refuse a refund at their sole discretion. Please refer to your proof of purchase, packing slip, or receipt to determine where you should return your product and the specific authorized dealers statements and policies. To request a replacement for a IntelliEAR®/PLUS unit, please return the unit with proof of purchase and a letter stating the reason for your return to: **IntelliEAR® Replacement Parts**, c/o KAGAN UNLIMITED, 3336 W LAWRENCE AVE SUITE 301, Chicago, IL 60625.

****Please Note:** Requests for replacement units or replacement parts submitted without proper payment will not be fulfilled. Please allow 4 weeks for processing all requests for replacements and replacement parts.*

PLEASE NOTE: THE INTELLIEAR® PERSONAL SOUND AMPLIFICATION PRODUCT IS INTENDED TO AMPLIFY ENVIRONMENTAL SOUND FOR NON-HEARING IMPAIRED USERS, AND IS NOT INTENDED TO COMPENSATE FOR HEARING IMPAIRMENT. IF YOU BELIEVE THAT YOUR HEARING IS IMPAIRED, YOU SHOULD CONSULT WITH A LICENSED PHYSICIAN.

Made in China

© 2015 KAGAN UNLIMITED, Chicago, IL 60625
U.S. PAT. NO. D632,794 All Rights Reserved.

www.intelliear.com